TGR-R KT 023B
TGR-R KT 023C

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BRUGSANVISNING

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Product code: TGR-R KT 023B / TGR-R KT 023C

1. Characteristics

Folding commode chair is made of powder coated steel frame. The model **TGR-R KT 023B** is equipped with height adjustable legs with anti-sliding rubber tips, while the model **TGR-R KT 023C** is supplied with legs with castors, the rear of which are equipped with brake. This foldable chair is equipped with toilet seat with lid and toilet bucket with lid which can be taken out easily. Owing to simple folding and adjustment, the chair can be adjusted to patients' needs and side armrests provide stability and safety use.

Elements of the commode chair:

- A. Armrests
- B. Toilet seat
- C. Toilet bucket
- D. Height adjustment
- E. Anti-sliding tips
- F. Castors (rear with brake)

2. Application

The commode chair is ancillary equipment which helps the user to perform physiological activities. The chair is recommended to people with dysfunctions and reduced mobility who have problems with unassisted walking, and for those who lost the ability of unassisted sitting down and standing up. The chair functions as a toilet owing to the toilet bucket placed under the seat. The chair facilitates performing physiological activities and taking care of the user. The castors enable comfortable sliding onto a toilet after removal of the toilet bucket.

3. Assembly

Unfolding and folding

- To unfold the chair, put it on the floor. Grab the plastic handles and unfold the chair. Assemble the front part of the frame.
- Assemble toilet seat with lid on the rear frame and toilet bucket in the middle.
- To fold the chair, remove toilet bucket and toilet seat with lid, lean downwards the front part of the frame, and then, lean one side forwards and the second side backwards.

Height adjustment

- To adjust the legs, press the metal pin on the leg, select appropriate height of the inner tube, and then, lock the position of the tubes relative to each other. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level. The legs must always stay stably on the floor.

Usage

Note! While getting on or off the chair, changing the user's position or being stationary, the brakes must be absolutely locked (**concerns TGR-R KT 023C**).

- Lift the toilet lid and remove the bucket lid. After use, cover the bucket with the lid and lower the toilet lid.

Note! Remember to empty the toilet bucket after each use.

4. Technical data

Overall width	56 cm
Inner width	46 cm
Overall depth	47 cm
Seat height from the ground TGR-R KT 023B	46-56 cm
Seat height from the ground TGR-R KT 023C	59-69 cm
Overall height from the ground TGR-R KT 023B	73-83 cm
Overall height from the ground TGR-R KT 023C	89-99 cm
Seat dimensions	38 cm x 40 cm
Chair weight TGR-R KT 023B	6.5 kg
Chair weight TGR-R KT 023C	8.0 kg
Maximum capacity	120 kg
Frame colour	grey

5. Notes

Timago International Group does not bear responsibility of improper use of the commode chair, failure to comply safety regulations and misuse.

6. Maintenance

Reviews

At least once every three months make a visual inspection of all the connecting components (latch, hinge) of the product including:

- the condition of the locking pins
- the condition of the castors and brakes
- stability and the look of the frame

Cleaning

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the toilet bucket and the toilet seat after each use. Use toilet cleaning agents according to producer's instructions.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

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